


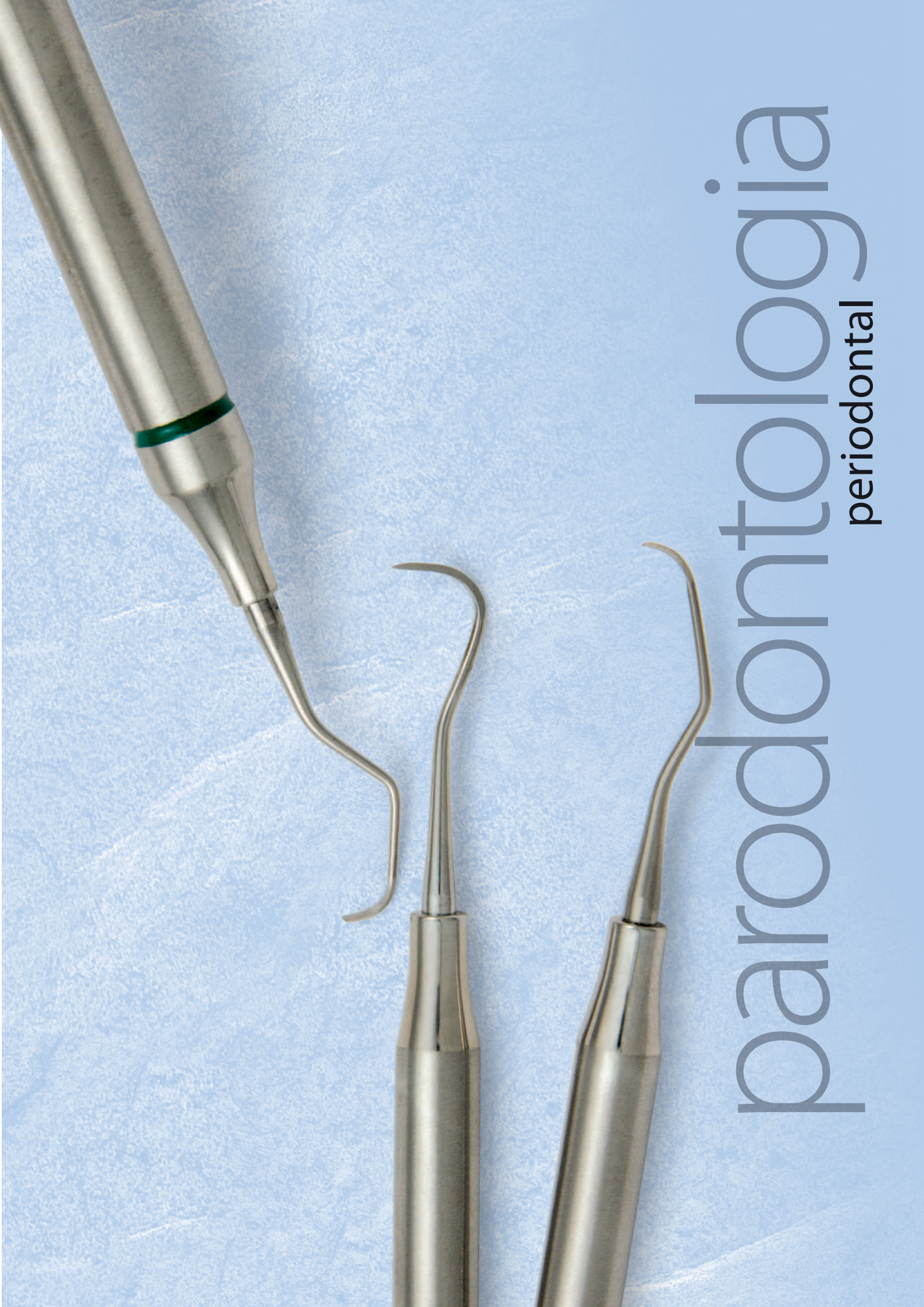

parodontologia

periodontal

strumenti per detartraggio | sickle scalers

Manico vuoto Ø 8 mm. - Hollow handle Ø 8 mm.



3...1	1...3
3...1	1...3

ZFM015
U15/33 - Towner-Jaquette



3...1	1...3
3...1	1...3

ZFM016
U15/30 - Towner-Jaquette



8...6	6...8
8...6	6...8

ZFM028
11/12 MC - McCall
Mesiale - Mesial



5...1	1...5
5...1	1...5

ZFM017
H5/33



8...6	6...8
8...6	6...8

ZFM030
13/14 MC - McCall
Distale - Distal



5...1	1...5
5...1	1...5

ZFM018
H6/H7



8...1	1...8
8...1	1...8

ZFM031
13S/14S MC - McCall



5...1	1...5
5...1	1...5

ZFM019
30/33 - Jaquette



8...6	6...8
8...6	6...8

ZFM032
17S/18S MC - McCall
Distale - Distal



8...1	1...8
8...1	1...8

ZFM023
T2/T3 - Taylor



8...1	1...8
8...1	1...8

ZFM035
M23



8...4	4...8
8...4	4...8

ZFM024
204



8...4	4...8
8...4	4...8

ZFM036
204SD



8...6	6...8
8...6	6...8

ZFM025
204S


curettes universali | universal curettes

Manico vuoto Ø 8 mm. - Hollow handle Ø 8 mm.




3...1	1...3
3...1	1...3

ZFM070
2R/2L C - Columbia

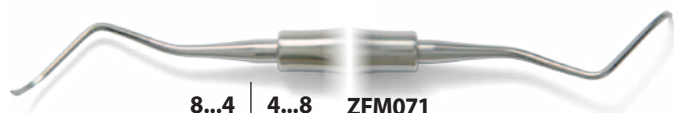

8...4	4...8
8...4	4...8

ZFM071
4R/4L C - Columbia

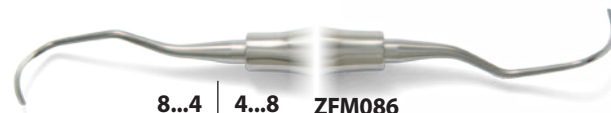

8...4	4...8
-	-

ZFM086
L 3/4 - Langer



8...4	4...8
8...4	4...8

ZFM072
13/14 C - Columbia



3...1	1...3
3...1	1...3

ZFM087
L 5/6 - Langer



8...6	6...8
8...6	6...8

ZFM073
YG 7/8 - Younger Good



8...1	1...8
8...1	1...8

ZFM088
M 23A




-	-
8...4	4...8

ZFM085
L 1/2 - Langer




8...1	1...8
8...1	1...8


ZFM089
M 23B

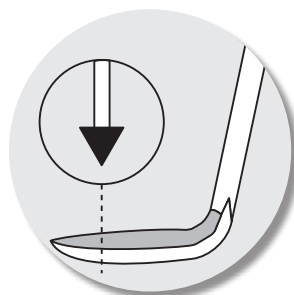

Gli **scaler a falce retto e curvo** si caratterizzano per:
- 2 bordi taglienti
- parte facciale ricurva
- estremità a punta

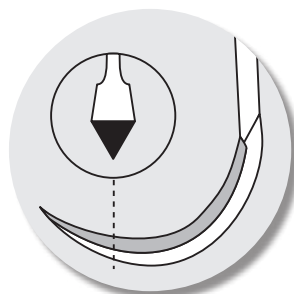


Straight and curved sickle scalers feature:
- 2 cutting edges
- curved facial component
- tipped ends

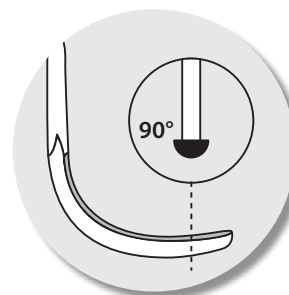

Le **curette universali** si caratterizzano per un angolo di 90° tra il gambo terminale e la parte tagliente.

Universal curettes are characterized by an angle of 90° between the terminal stem and the cutting component.

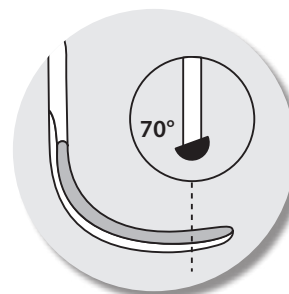

Le **curette di Gracey** si caratterizzano per un angolo di 70° tra il gambo terminale e la parte tagliente.

Gracey curettes are characterized by an angle of 70° between the terminal stem and the cutting component.

curettes di Gracey | Gracey curettes

Manico vuoto Ø 8 mm. - Hollow handle Ø 8 mm.



3...1	1...3
3...1	1...3

ZFM100

1/2 GR - Gracey

Superfici vestibolari

Buccal surfaces



5...1	1...5
5...1	1...5

ZFM101

3/4 GR - Gracey

Superfici linguali/palatali

Lingual/palatal surfaces



5...1	1...5
5...1	1...5

ZFM102

5/6 GR - Gracey



8...4	4...8
8...4	4...8

ZFM103

7/8 GR - Gracey

Superfici vestibolari e linguali/palatali

Buccal and lingual/palatal surfaces



8...4	4...8
8...4	4...8

ZFM104

9/10 GR - Gracey

Superfici vestibolari e linguali/palatali -

Più angolata

Buccal and lingual/palatal surfaces -

More angled



8...4	4...8
8...4	4...8

ZFM105

11/12 GR - Gracey

Mesiale - Mesial




8...4	4...8
8...4	4...8

ZFM106

13/14 GR - Gracey

Distale - Distal

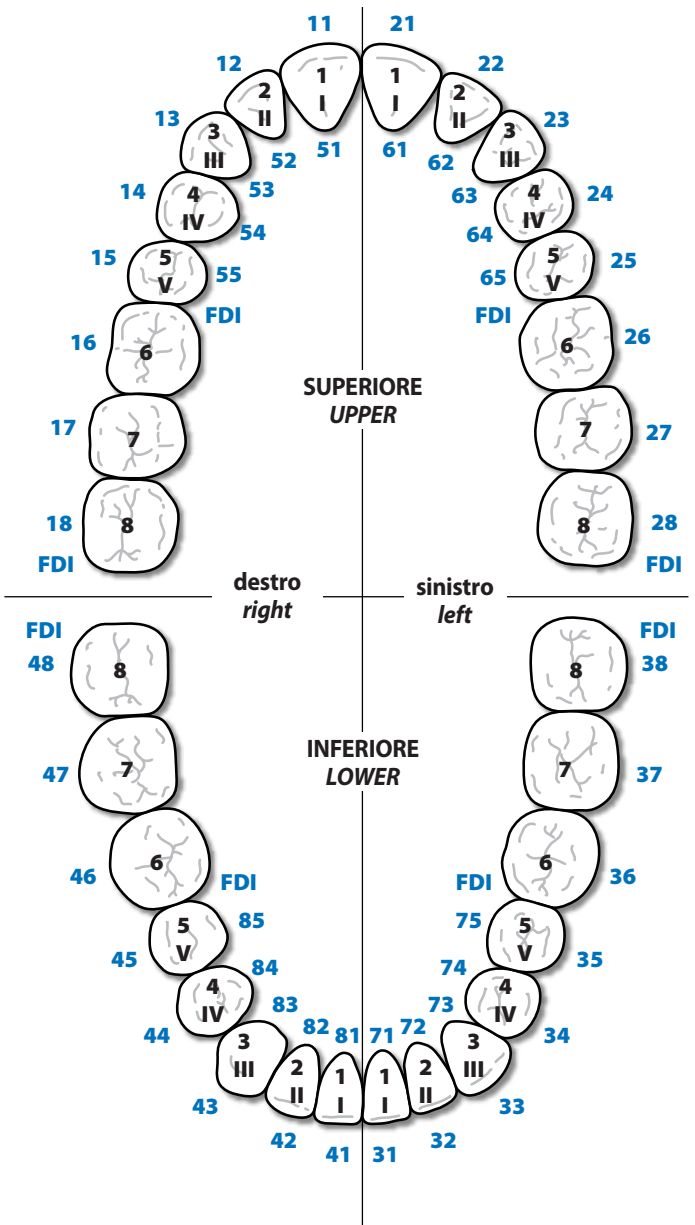

curettes di Gracey | Gracey curettes

Manico vuoto Ø 10mm. - Hollow handle Ø 10mm.



3...1	1...3
3...1	1...3

ZFM200

1/2 GR - Gracey

Superfici vestibolari

Buccal surfaces



5...1	1...5
5...1	1...5

ZFM201

3/4 GR - Gracey

Superfici linguali/palatali

Lingual/palatal surfaces



5...1	1...5
5...1	1...5

ZFM202

5/6 GR - Gracey



8...4	4...8
8...4	4...8

ZFM205

11/12 GR - Gracey

Mesiale - Mesial



8...4	4...8
8...4	4...8

ZFM206

13/14 GR - Gracey

Distale - Distal



8...4	4...8
8...4	4...8

ZFM203

7/8 GR - Gracey

Superfici vestibolari e linguali/palatali

Buccal and lingual/palatal surfaces



8...4	4...8
8...4	4...8

ZFM207

15/16 GR - Gracey

Mesiale - Mesial



8...4	4...8
8...4	4...8

ZFM204

9/10 GR - Gracey

Superfici vestibolari e linguali/palatali -

Più angolata

Buccal and lingual/palatal surfaces -

More angled




8...4	4...8
8...4	4...8


ZFM208

17/18 GR - Gracey

Distale - Distal

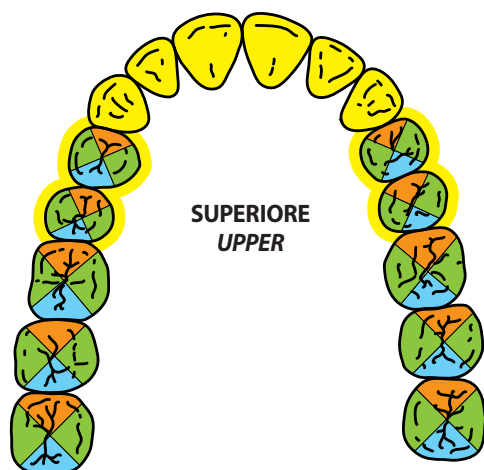

SUPERIORE
UPPER

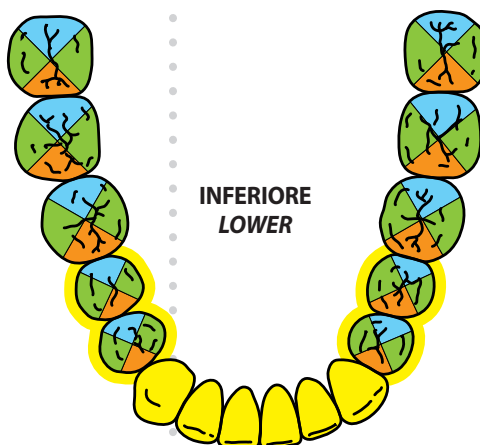

INFERIORE
LOWER

- # 1/2 - # 3/4 - # 5/6
- # 7/8 - # 9/10
- # 11/12 - # 15/16
- # 13/14 - # 17/18

curettes mini Gracey | Gracey mini-curettes

Manico vuoto Ø 8 mm. - Hollow handle Ø 8 mm.



3...1	1...3
3...1	1...3

ZFM140
1/2 GRL 3M - Gracey



5...1	1...5
5...1	1...5

ZFM141
3/4 GRL 3M - Gracey



8...4	4...8
8...4	4...8

ZFM145
11/12 GRL 3M - Gracey
Mesiale - Mesial



5...1	1...5
5...1	1...5

ZFM142
5/6 GRL 3M - Gracey




8...4	4...8
8...4	4...8

ZFM146
13/14 GRL 3M - Gracey
Distale - Distal



8...4	4...8
8...4	4...8

ZFM143
7/8 GRL 3M - Gracey
Superfici vestibolari e linguali/palatali
Buccal and lingual/palatal surfaces

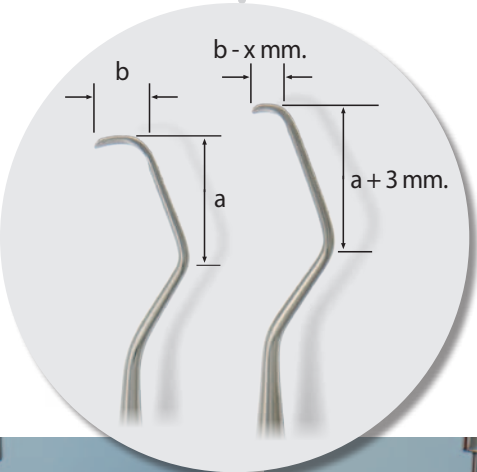

curettes universali in titanio | titanium universal curettes

Specifiche per utilizzo su impianti - *For implants use*



ZFM685 new

L 1/2 - Langer



ZFM686 new

L 3/4 - Langer



ZFM687 new

L 5/6 - Langer



pietre per affilatura | sharpening stones

Le pietre per affilatura LASCOD sono da utilizzare per affilatura manuale. In quanto pietre naturali sono possibili venature ed irregolarità cromatiche. Utilizzare pietre a grana media per affilare strumenti molto smussati o per strumenti non correttamente affilati. Dopo aver utilizzato pietre a grana media è necessario ripassare con pietre a grana fine. Utilizzare pietre a grana fine per mantenere l'affilatura o per rimuovere piccole sbavature.

Per l'affilatura si consiglia di utilizzare alcune gocce di olio.

MANUTENZIONE DELLE PIETRE DA AFFILATURA

Durante l'affilatura dello strumentario i residui metallici asportati tendono ad ostruire i pori delle pietre. Si consiglia di pulire periodicamente le pietre con alcune gocce di olio lubrificante spalmandolo sulla superficie. Rimuovere i residui metallici con una garza.

Prima di procedere con le operazioni di sterilizzazione delle pietre si consiglia di pulirle con ultrasuoni o con una spazzola non metallica.

Sharpening stones by LASCOD are to be used for manual sharpening. As natural stones they may present veins and variations in colour. Use medium-grain stones to sharpen blunt instruments or incorrectly sharpened instruments. After using medium-grain stones, then polish instrument with fine-grain stones. Use fine-grain stones to preserve the sharpening or remove small burrs. Using a few drops of oil on the stone while sharpening is recommended.

MAINTENANCE OF SHARPENING STONES

During the sharpening of the instruments the removed metal debris may clog the pores of the stones. It is recommended to clean the stones periodically by rubbing of a few drops of lubricating oil upon the surface of the stone. Remove the metal debris with dry gauze. It is recommended to clean the stones with ultrasonic or with a non-metallic brush before carrying out any sterilization procedures.



ZFM900

6A - Arkansas

2/7 x 40 x 100 mm

Pietra Arkansas grana extra fine (1200)

Arkansas stone extra fine grain (1200)



ZFM901

AS22 - Arkansas

0,5/5 x 25 x 60 mm

Pietra Arkansas grana extra fine (1200)

Arkansas stone extra fine grain (1200)



ZFM902

6 - Corindone

2/10 x 50 x 110 mm

Pietra India grana media (220)

Indian stone medium grain (220)



ZFM903

1 - Tiger

7 x 32 x 80 mm

Pietra naturale grana media (250)

Natural stone medium-grain (250)

pietre per affilatura | sharpening stones



ZFM904
2 - Tiger
7 x 32 x 80 mm
Pietra naturale grana fine (500)
Natural stone fine-grain (500)



ZFM905 **new**
3 - Arkansas
6 x 25 x 85 mm
Pietra Arkansas grana extra fine (1200)
Arkansas stone extra fine grain (1200)



ZFM906 **new**
4 - Arkansas
Ø 8 x 100 mm
Pietra Arkansas cilindrica grana extra fine (1200)
Cylindrical Arkansas stone extra fine grain (1200)

accessori per affilatura | sharpening accessories

ZFM930
Olio per affilatura
strumenti
*Oil for instrument
sharpening*



ZFM920
Bastoncino per test
affilatura
Testing stick



X-Red curettes di Gracey | X-Red Gracey curettes



Manico vuoto Ø 10 mm. - Hollow handle Ø 10 mm.



3...1	1...3
3...1	1...3

ZFM800
1/2 GR - Gracey



5...1	1...5
5...1	1...5

ZFM801
3/4 GR - Gracey
Superfici linguali/palatali
Lingual/palatal surfaces



8...4	4...8
8...4	4...8

ZFM804
9/10 GR - Gracey
Superfici vestibolari e linguali/palatali -
Più angolata
*Buccal and lingual/palatal surfaces -
More angled*



5...1	1...5
5...1	1...5

ZFM802
5/6 GR - Gracey



8...4	4...8
8...4	4...8

ZFM805
11/12 GR - Gracey
Mesiale - Mesial



8...4	4...8
8...4	4...8

ZFM803
7/8 GR - Gracey
Superfici vestibolari e linguali/palatali
Buccal and lingual/palatal surfaces



8...4	4...8
8...4	4...8

ZFM806
13/14 GR - Gracey
Distale - Distal

X-Red strumenti per detartraggio | X-Red sickle scaler

Manico vuoto Ø 10 mm. - Hollow handle Ø 10 mm.



5...1	1...5
5...1	1...5

ZFM817
H 5/33



5...1	1...5
5...1	1...5

ZFM818
H 6/7



8...6	6...8
8...6	6...8

ZFM825
204 S



8...1	1...8
8...1	1...8

ZFM823
T2/T3 - Taylor



8...4	4...8
8...4	4...8

ZFM836
204 SD

Per maggiori informazioni su X-Red consultare pagg. 10-13

For more information on X-Red see pages 10-13